

# Elevation Diagnostics Test Kit Instructions

Go to the Kit Registration page by scanning the QR code with your phone's camera or typing the web address. Enter the clinic code provided at kit receipt. Complete all required fields and check for a "Completed!" message when finished. You will receive a text and email confirmation.



<https://elevationdx.mytests.io/#/patient/register/site-selection>

**!** Please wait 30 minutes after ingesting any substance including food, water, or inhalants to avoid contaminating your sample

Fill the tube with 1 to 2 mL of liquid saliva (about a quarter of the tube full). Do not cough or sniff prior to providing your sample as this can, also, cause contamination and delay or prohibit the processing of your results.



Screw the cap securely onto the sample tube. Write your date of birth on the included labels, then apply one to the tube and one to the outside of the biohazard bag. Place your sample tube into the biohazard bag and close the bag.



Give your sample to the testing coordinator for confirmation. You will be notified via text and email when the lab receives your sample and again when your results are ready. For any questions about this test, please visit [www.elevationdiagnostics.com/patientFAQ](http://www.elevationdiagnostics.com/patientFAQ). Thank you for testing with Elevation Diagnostics.



Questions? Call 800-440-5184

  @ElevationDiagnostics